

INDIVIDUAL CUSTOMISABLE POSITIONING SOLUTIONS FOR THE PELVIS P.T.S. (The Pelvic Total Support)

POSTURE FOR PASSION FOR OVER 20 YEARS

REAL ANSWERS TO SPECIAL NEEDS

Years of experience working together with users, clinicians, technicians and care givers to develop postural solutions answering to the most complex needs.





Our researches and works have been accepted in important congresses both in Italy and abroad stating our

PTS (PELVIC TOTAL SUPPORT) A NEW APPROACH TO THE POSTURE OF PELVIS

CONCEPT:

Total support of the Femurs/Pelvis/Spine relationship system (coccygeal/sacral/lumbar) of Men, Women and Children.









METHODOLOGY:

Balancing of the Femurs/Pelvis/Spine relationship system (coccyx, sacrum, lumbar) containment and stabilisation of the anatomy, also asymmetric.



Test Results

The distribution of the pressures fluctuates within a range of between 0,20 PSI and 3,87 PSI (9,83 mmHg and 126,79 mmHg), the coloured areas highlight the distribution of pressure peaks on the cushion. The blue area indicates a uniformly distributed area surrounding the pelvis.

TECHNOLOGY:

Modular structure composed of a flat base of construction and independent pre-shaped inserts, assembled to fit every shape, size and morphology of the seated person.







EASY LIKE A PUZZLE









For partially sighted users this document can be visualised in PDF format on the web site www.versainserto.com with the possibility to enlarge the text.

POSTURE FOR PASSION FOR OVER 20 YEARS

A WORLD IN A CUSHION

PERFECT WEREABILITY THAT FITS:

- man, woman, child pelvis
- different and combined deformities
- various morphologies for the whole life cycle of use







EACH INSERTO MODEL CONFIGURATION INCLUDES AS STANDARD:



MODULAR INSERTS KIT



MODULAR PADDING KIT



COVER

STRUCTURE AND MATERIALS TO ALLOW ALIGNMENT, POSTURAL CONTROL, LOADS DISTRIBUTION AND COMFORT

STRUCTURAL KIT for the individual seating construction, set up and prepared on the user



Different densities combination of polyethylene foam.

It absorbs the vibrations and provides a slow memory return to deformations. Extremely lightweight and easy to modify by cutter.

Free Shape PADDING KIT to ensure a perfect adherence to the structural kit, the body and movements



Flexible elastic expanded polyurethane foam that supports the movement.

Allows air circulation.



Flexible open celled visco-elastic polyurethane foam.

Anti-vibration and shock-absorbing.

FOR OVER 15

MATERIALS

TESTED

Anti-decubitus with low spring back action polyurethane foam. Helps in the distribution of pressures and matches the body shape.

NEW 3D" COVER



- ELASTIC - INCONTINENT - AIR EXCHANGE - WASHABLE AT 60°C



Stable, safe and durable: The cushions passed the repetitive load testing required by Iso 16840-3:2014

TESTED

Fireproof: The cushions meet the flammability requirements EN 1021-1:2014 and EN 1021-2:2014


INDIVIDUAL CUSTOMISABLE POSITIONING SOLUTIONS FOR THE PELVIS









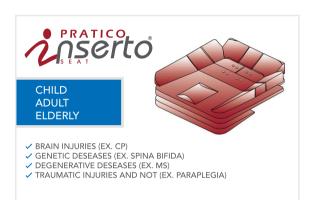













(EX. MD, LEUKODISTROPHY, SMA, RETT SYNDROME)



MODEL		BASICO	MODO	PRATICO	IMPULSO	MEDIANO	DOMINO	NAUCO	NOVO	TANTO	PUCCO
						Comp. To					
USER	SELF PROPELLING ABILITY	YES / NO	YES	YES	YES / NO	NO	YES / NO	YES / NO	YES / NO	YES / NO	YES / NO
	RISK OF PRESSURE LOAD	AVERAGE/HIGH	AVERAGE	LOW	HIGH	AVERAGE	HIGH	HIGH	HIGH	HIGH	HIGH
DEFORMITIES	ALIGNMENT CAPACITY	LOW	LOW/MODERATE	LOW	MODERATE/HIGH	HIGH	MODERATE/HIGH	MODERATE/HIGH	MODERATE/HIGH	MODERATE/HIGH	MODERATE/HIGH
	CONTAINMENT CAPACITY	LOW	LOW/MODERATE	LOW	MODERATE	MODERATE	HIGH	HIGH	HIGH	HIGH	HIGH
FEATURES	EASY LATERAL TRANSFERS	SUITABLE	SUITABLE	SUITABLE	SUITABLE	LESS SUITABLE	UNSUITABLE	UNSUITABLE	SUITABLE	SUITABLE	UNSUITABLE





MANUFACTURED BY



Pro Medicare srl

Via Montagna Z.I. - Lotto 41 72023 Mesagne (BR) - ITALY Tel. +(39) 0831 777840 Fax. +(39) 0831 730739 Email: sales@promedicare.it www.versainserto.com www.promedicare.eu

CERTIFIED COMPANY WITH QUALITY MANAGEMENT SYSTEM ISO 9001 / EN ISO 13485 N° 279412 / N° 279411









DISTRIBUTED BY:





C€ 2 YEAR WARRANTY

www.versainserto.com